



BREAST PROSTHESIS examples



Post Surgery
Feather Weight



Harmony Silk Triangle
Lightweight



Amica Soft
Lightweight



E-Supreme
Traditional



A-Supreme
Traditional



Symphony
Traditional



Swimming
Calypso

How do I clean my prosthesis?

Wash your prosthesis daily:

- You will take a small amount (the size of a pea) of antimicrobial soap and put it onto a wet cloth. Use this to wipe down the breast prosthesis until it foams. Please clean the front and the back of the prosthesis
- You can also completely submerge the breast prosthesis in lukewarm water and soap
- You can then wipe with a dry cloth

It is also important to note that you should avoid the following application of products to the chest wall as well as the Breast Prosthesis:

Powder / Deodorant / Perfume / Moisturiser / Tanning products / Lotions

For more information please contact one of our Orthotist/Prosthetist at one of our branches closest to you.